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- ¹ Chen ST, Wilson TG Jr, Hämmerle CH. Immediate or early placement of implants following tooth extraction: review of biologic basis, clinical procedures, and outcomes. *Int J Oral Maxillofac Implants* 2004; 19 Suppl: 12–25.
- ² Chen ST, Darby IB, Reynolds EC. A prospective clinical study of non-submerged immediate implants. Clinical outcomes and esthetic results. *Clin Oral Implants Res* 2007; 18: 552–562.
- ³ Buser D, Weber HP, Lang NP. Tissue integration of non-submerged implants. 1-year results of a prospective study with 100 ITI hollow-cylinder and hollow-screw implants. *Clin Oral Implants Res* 1990; 1: 33–40.
- ⁴ Bornstein MM, Schmid B, Belser UC, Lussi A, Buser D. Early loading of non-submerged titanium implants with a sandblasted and acid-etched surface. 5-year results of a prospective study in partially edentulous patients. *Clin Oral Implants Res* 2005; 16: 631–638.
- ⁵ Weber HP, Crohin CC, Fiorellini JP. A 5-year prospective clinical and radiographic study of non-submerged dental implants. *Clin Oral Implants Res* 2000; 11: 144–153.
- ⁶ Behneke A, Behneke N, D'Hoedt B. The longitudinal clinical effectiveness of ITI solid-screw implants in partially edentulous patients. *Int J Oral Maxillofac Implants* 2000; 15: 633–645.
- ⁷ Lindeboom JA, Tjiook Y, Kroon FH. Immediate placement of implants in periapical infected sites: a prospective randomized study in 50 patients. *Oral Surg Oral Med Oral Pathol Oral Radiol Endod* 2006; 101: 705–710.
- ⁸ Chen ST, Darby IB, Reynolds EC. A prospective clinical study of non-submerged immediate implants: clinical outcomes and esthetic results. *Clin Oral Implants Res* 2007; 18: 552–562.
- ⁹ Evans CD, Chen ST. Esthetic outcomes of immediate implant placements. *Clin Oral Implants Res* 2008; 19: 73–80.

www.straumann.com

International Headquarters

Institut Straumann AG
 Peter Merian-Weg 12
 CH-4002 Basel, Switzerland
 Phone +41 (0)61 965 11 11
 Fax +41 (0)61 965 11 01